

Features

- Large red call button to trigger calls
- Red LED for call indication
- Includes replaceable BA501 battery
- Includes lanyard with anti-ligature point
- Optional RP571 wristband
- A reset signal is triggered using the RP570 Caregiver Reset Key
- Waterproof to IP65

Description

The SF570 Pendant Transmitter features the great range and reliability of 433 MHz in a small waterproof package. The pendants can be worn in the shower or bathtub, outside in the rain, or other environments where they may be splashed with water or momentarily submerged.

The pendants have a single-button design with “press and one-second hold” activation, making them easy to use but resistant to false alarms. When a pendant is activated its red LED stays solid for five seconds, then blinks slowly until the alarm is cleared. Alarms may be cleared by placing an RP570 Alarm Clearance Card over the button.

The pendant transmitters are especially suited for assisted living installations. They feature a fully supervised RF link to ensure reliability. The built-in LED flashes with all transmissions, including supervision, so the user can see that the transmitter is still functioning.

Required Components

NC570 Receiver
NC571 Repeater
RP570K Caregiver Reset Key 10-piece set

Optional Components

RP571K Wristband option



SF570 Pendant Transmitter
SF570 with Wristband Option
RP570 Caregiver Reset Key

Specifications

Dimensions:	2"H × 1.375"W × 0.625"D (51 mm × 35 mm × 16 mm)
Construction:	White plastic
Frequency:	433.92 MHz
Battery:	BA501 (CR2450N)

TekTone® has designed Tek-CARE®570 to work as a system. Its Tek-CARE® Appliance Server has been carefully designed and matched to work with Tek-CARE570 wireless and wired peripherals. TekTone will not warrant or support components of Tek-CARE570 that are purchased separately unless the components are add-ons to previously sold systems. TekTone will also not warrant or support a system that is a mix of TekTone and non-TekTone parts, unless prior permission has been obtained. A site survey must be performed and submitted before ordering a Tek-CARE570 system. In order to track our systems and to inform facilities of software upgrades, full site information (site name, address and phone number) will be required when ordering. Site information and Tek-CARE Appliance Server warranty numbers will also be required when requesting technical support.