



NC369 UHF Paging Transmitter Specification Sheet

IL670
Section E
Rev. 8 - 08/2010



NC369

ARCHITECTS' AND ENGINEERS' SPECIFICATION

The UHF on-site paging system transmitter/control unit shall be TekTone® NC369 or approved equal. The NC369 shall support one Tek-CARE® NC300II, Tek-CARE®400 or Tek-CARE®500 system. The paging information shall be supplied to the unit at 9600 baud and shall be transmitted to the radio pagers at 1200 baud on 457.550 Mhz. Information such as call type, location and priority shall be sent to the pagers in its entirety. LEDs shall indicate transmitter activity.

The paging system shall support up to 1000 pager IDs and shall have adjustable transmission strength of up to two watts (site survey recommended).

Paging is an ancillary method of call annunciation, and is not intended to replace the primary annunciation provided by the nurse call system.

FEATURES

The NC369 UHF Paging Transmitter receives alphanumeric messages and patient call information via serial interface and transmits the information directly to pagers. Information is sent to the transmitter at 9600 baud to minimize bottlenecks. Call type and location are determined by the system supplying information to the transmitter. Up to 1000 pager IDs are supported. Battery backup is also available. Programming is simple using a PC or laptop and web based programming screens. Dipswitches are also available on the unit for minimal function onsite setup.

The system's range depends upon the site and the transmitter's location. To determine the exact coverage, conduct a site survey. Estimate a range of up to one mile under ideal conditions. The unit includes a ¼-wave BNC-terminated antenna to provide coverage that meets most requirements. The optional CA133 Antenna is suited for installations that require good building penetration and range. On average, the CA133 Antenna adds about 25% to the transmission footprint, and because it comes with approximately 10 feet of cable, it can be mounted in more locations.

SPECIFICATIONS

- Dimensions: 2.7" × 10" × 8"
(6.86 cm × 25.4 cm × 20.32 cm)
- Construction: Electronic components mounted on a PCB enclosed in an ABS housing
- Connections: Plug-in RJ45 to DB9
- Housing: Surface or shelf mounted
- Frequency: Programmable for 440–470 MHz
- RF Power Output: Up to 2 watts
- Voltage: 12VDC at 2A Input 115 VAC
- FCC ID No.: T5GPT5
- IC (Canada): IC4767A-PT5
- BAUD Rate: 9600 serial input, 1200 transmit

REQUIRED COMPONENTS

- NC399P Pagers
- Tek-CARE®400 Nurse Call System
- or- Tek-CARE®NC300II Nurse Call System
- or- Tek-CARE®NC110/150/200 Nurse Call System
(requires NC375 Transmitter Interface)
- or- Tek-CARE®500 Wireless Emergency Call System

www.tektone.com

277 Industrial Park Road • Franklin, NC 28734 • tektone@tektone.net
Phone: (828) 524-9967 • Fax: (828) 524-9968 • Tech Support: Choose option 2 • Sales: Choose option 3
TekTone®'s quality system is registered by UL® to the ISO 9001 standard. (Reference #10001510.)