

DRAWING LIST


- NC0 NURSE CALL COVER PAGE
NC1 1ST FLOOR NURSE CALL PLAN
NC2 NURSE CALL SYSTEM RISER DIAGRAM



Ann@PacelliAssociates.com
(518) 414-4302
NYS WBE Certified
PacelliAssociates.com




TekTone 20-Bed Nurse Call System Using
Tek-CARE 3000

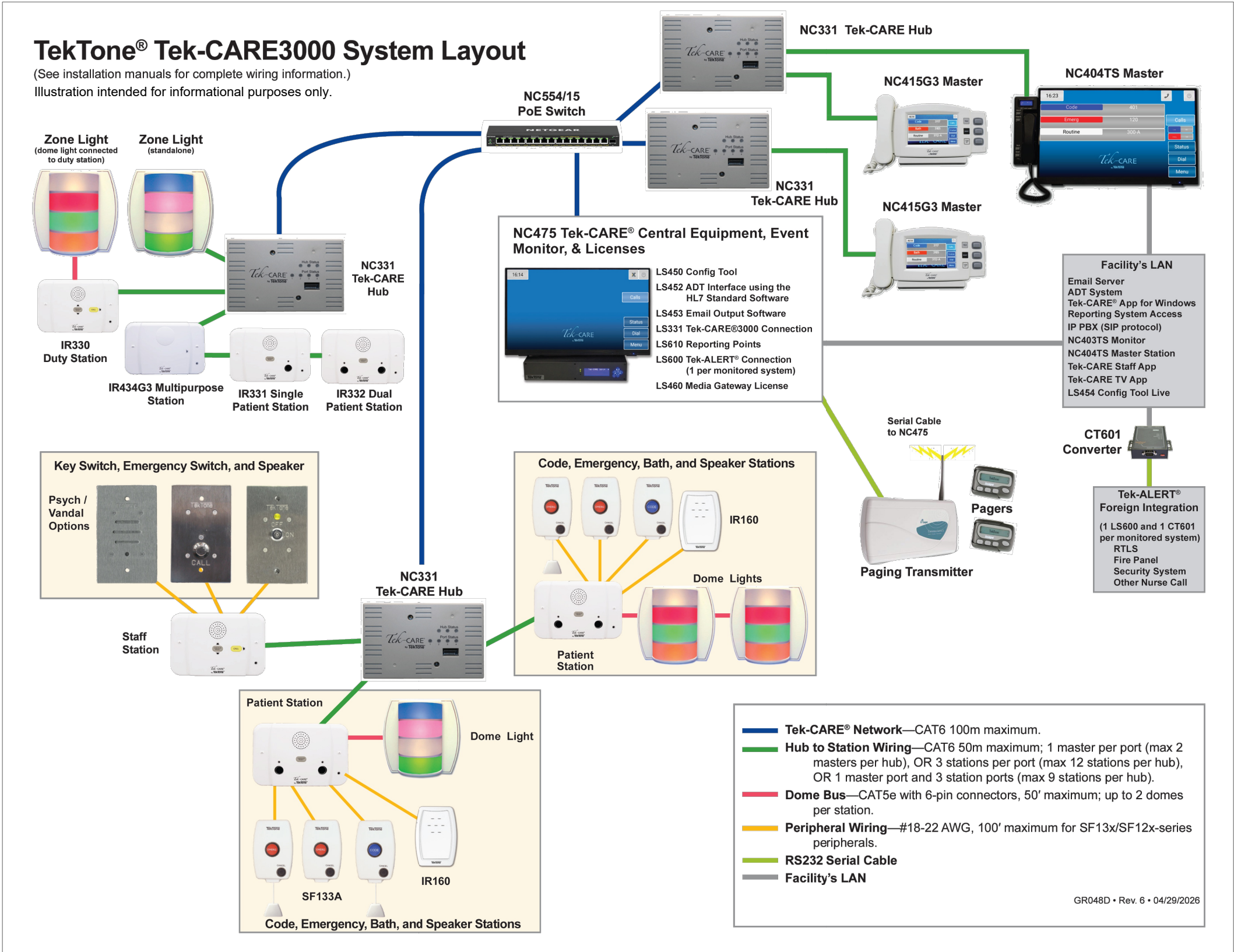

REVISIONS		DATE
No.	DESCRIPTION	
0	FINAL TO TEK TONE	04/30/28
1	REVISION 01 TO TEK TONE	05/07/28
2	REVISION 02 TO TEK TONE	05/17/28
3	REVISION 03 TO TEK TONE	05/17/28

DRAWING TITLE:
NURSE CALL
COVER
PAGE

DRAWING NUMBER:
NC0

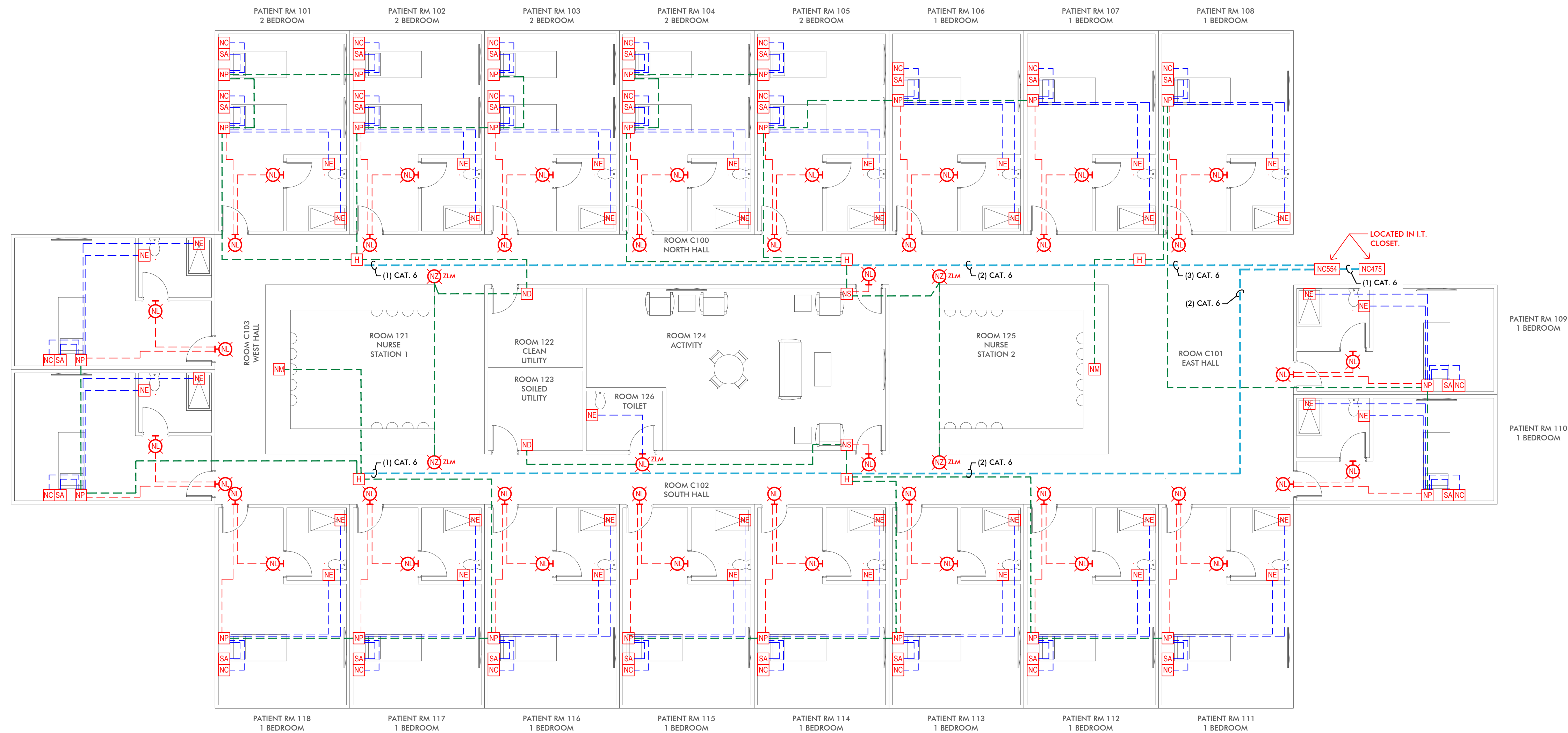

FLOOR PLAN
SCALE: 1/8" = 1'-0"



LEGEND

- NM** MASTER STATION - WALL OR DESK MOUNTED
OPTION #1: CONSOLE TYPE WITH HANDSET:
- TekTONE NC415G3 WITH TA415AL HANDSET
- TekTONE IH415LD DESK MOUNT HOUSING OR IH415W WALL MOUNT BRACKET
OPTION #2: 24" TOUCH SCREEN MONITOR TYPE WITH HANDSET:
- TekTONE NC404TS
- WALL MOUNT USES STANDARD VESA MOUNTING SYSTEM (PROVIDED BY CONTRACTOR)
- NE** EMERGENCY STATION - 48" AFF (72" AFF WHEN LOCATED IN SHOWER)
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER RING
- BOTTOM OF PULL TO BE 2" AFF IF STATION CONTAINS A PULL CORD
- TekTONE SF133A
- PROVIDE TekTONE SF133GK GASKET KIT WHEN MOUNTED IN SHOWER
- NP** SINGLE PATIENT STATION - 48" AFF
- MOUNTED IN 2-GANG OR 3-GANG 2-1/2" DEEP BOX WITH PLASTER RING
- TekTONE IR301
- TekTONE SF301B 8' CALL CORD
- NC** CODE BLUE BUTTON - 48" AFF
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER RING
- TekTONE SF133A
- SA** STAFF ASSIST BUTTON - 48" AFF
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER RING
- TekTONE SF133A
- NS** AUDIO VISUAL STAFF STATION - 48" AFF
- MOUNTED IN 2-GANG OR 3-GANG 2-1/2" DEEP BOX WITH PLASTER RING
- TekTONE IR330
- ND** DUTY STATION - 48" AFF
- MOUNTED IN 2-GANG OR 3-GANG 2-1/2" DEEP BOX WITH PLASTER RING
- TekTONE IR330
- NL** CORRIDOR DOME LIGHT - 84" AFF
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER RING OR A 2-GANG BOX WITH A 1-GANG PLASTER RING
- TekTONE LI484P5
- NL ZLM** CORRIDOR DOME LIGHT STATION - 84" AFF
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER RING OR A 2-GANG BOX WITH A 1-GANG PLASTER RING
- LIGHT TO CONTAIN A ZONE LIGHT MODULE
- TekTONE LI484P5 WITH PM424ZP5 ZONE LIGHT MODULE
- NZ ZLM** CORRIDOR ZONE LIGHT STATION - CEILING MOUNTED
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER RING OR A 2-GANG BOX WITH A 1-GANG PLASTER RING
- ZONE LIGHT TO CONTAIN A ZONE LIGHT MODULE
- TekTONE LI484P5 WITH PM424ZP5 ZONE LIGHT MODULE
- H** Tek-CARE HUB
- MOUNTED IN 2-GANG ELECTRICAL BOX ABOVE ACCESSIBLE CEILING
- TekTONE NC331
- NC475** Tek-CARE SERVER, RACK MOUNTED
- TekTONE NC475 (TWO U RACK SPACES)
- NC554** Tek-CARE 15 PORT NETWORK SWITCH
- WALL MOUNTED OR DESK MOUNTED
- TekTONE NC554/15

WIRE LEGEND

- COMMUNICATION BUS (C-BUS) HOMERUN TO POE SWITCH. EACH HUB REQUIRES A DEDICATED HOMERUN**
(1) CATEGORY 5e OR BETTER
MAXIMUM LENGTH = 328'
8-PIN CONNECTOR REQUIRED.
- HUB STATION WIRING**
(1) CATEGORY 5e OR BETTER
MAXIMUM LENGTH = 175'
8-PIN CONNECTOR REQUIRED.
- PHONE BUS**
(1) CATEGORY 5e OR BETTER
MAXIMUM LENGTH = 50'
6-PIN CONNECTOR REQUIRED.
- PERIPHERAL WIRING**
(2) CONDUCTOR #18-#22 STRANDED
MAXIMUM LENGTH = 100'



Ann@PacelliAssociates.com
(518) 414-4302
NYS WBE Certified
PacelliAssociates.com




TekTone 20-Bed Nurse Call System Using
Tek-CARE 3000

REVISIONS		DATE
No.	DESCRIPTION	
0	FINAL TO TEK TONE	04/03/20
1	REVISION 01 TO TEK TONE	05/07/20
2	REVISION 02 TO TEK TONE	05/12/20
3	REVISION 03 TO TEK TONE	05/12/20

DRAWING TITLE:
1ST FLOOR
NURSE CALL
PLAN

DRAWING NUMBER:
NC1

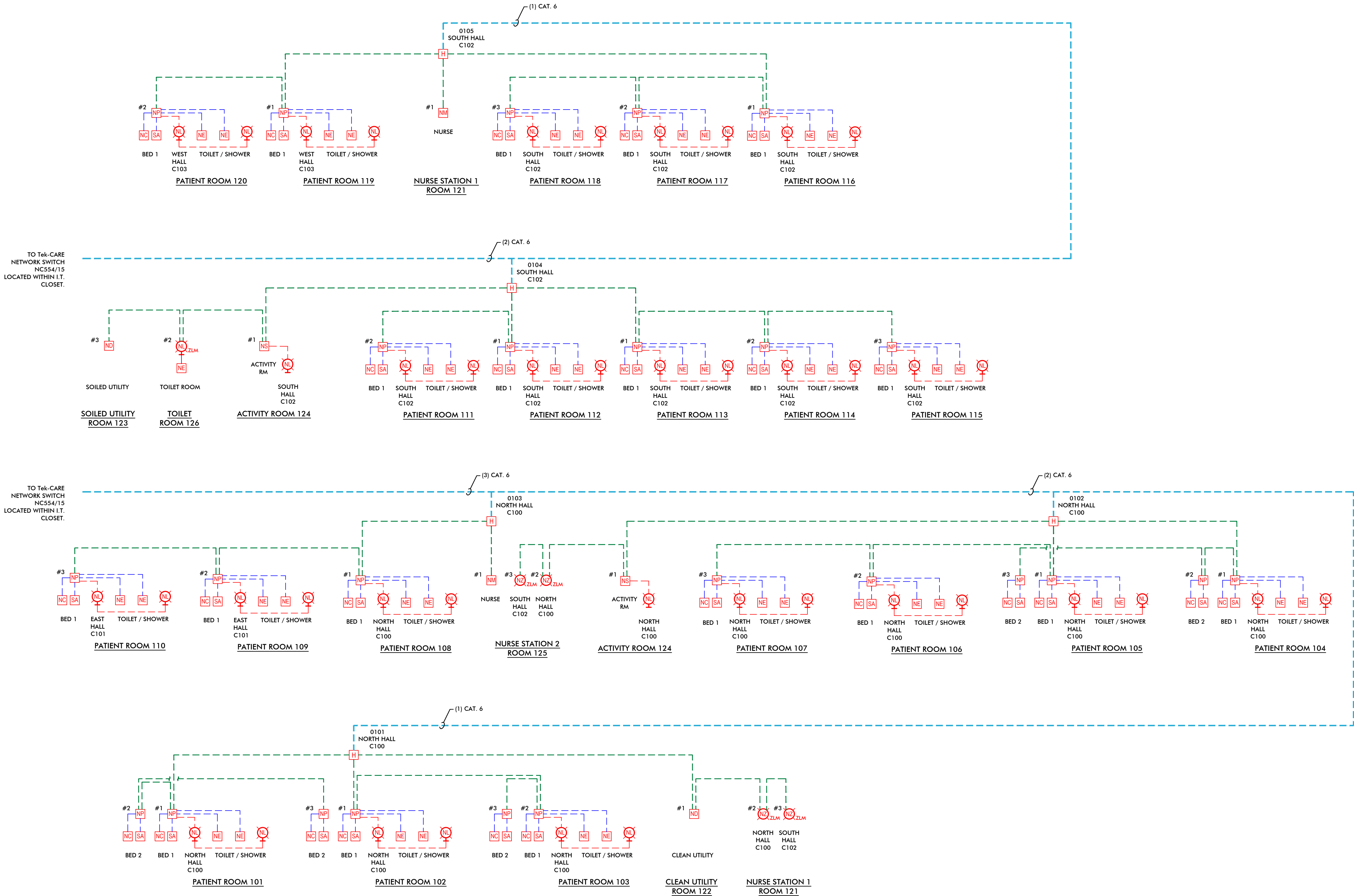

ONE LINE DIAGRAM
SCALE: NONE

LEGEND

- NM** MASTER STATION - WALL OR DESK MOUNTED
OPTION #1: CONSOLE TYPE WITH HANDSET:
- TekTONE NC415G3 WITH TA415AL HANDSET
- TekTONE IH415LD DESK MOUNT HOUSING OR
IH415W WALL MOUNT BRACKET
OPTION #2: 24" TOUCH SCREEN MONITOR TYPE WITH
HANDSET:
- TekTONE NC404TS
- WALL MOUNT USES STANDARD VESA MOUNTING
SYSTEM (PROVIDED BY CONTRACTOR)
- NE** EMERGENCY STATION - 48" AFF (72" AFF WHEN
LOCATED IN SHOWER)
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER
RING
- BOTTOM OF PULL TO BE 2" AFF IF STATION
CONTAINS A PULL CORD
- TekTONE SF133A
- PROVIDE TekTONE SF133GK GASKET KIT WHEN
MOUNTED IN SHOWER
- NP** SINGLE PATIENT STATION - 48" AFF
- MOUNTED IN 2-GANG OR 3-GANG 2-1/2" DEEP
BOX WITH PLASTER RING
- TekTONE IR331
- TekTONE SF301B 8" CALL CORD
- NC** CODE BLUE BUTTON - 48" AFF
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER
RING
- TekTONE SF133A
- SA** STAFF ASSIST BUTTON - 48" AFF
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER
RING
- TekTONE SF133A
- NS** AUDIO VISUAL STAFF STATION - 48" AFF
- MOUNTED IN 2-GANG OR 3-GANG 2-1/2" DEEP
BOX WITH PLASTER RING
- TekTONE IR330
- ND** DUTY STATION - 48" AFF
- MOUNTED IN 2-GANG OR 3-GANG 2-1/2" DEEP
BOX WITH PLASTER RING
- TekTONE IR330
- H** CORRIDOR DOME LIGHT - 84" AFF
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER
RING OR A 2-GANG BOX WITH A 1-GANG PLASTER
RING
- TekTONE LI484P5
- H ZLM** CORRIDOR DOME LIGHT STATION - 84" AFF
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER
RING OR A 2-GANG BOX WITH A 1-GANG PLASTER
RING
- LIGHT TO CONTAIN A ZONE LIGHT MODULE
- TekTONE LI484P5 WITH PM424ZP5 ZONE LIGHT
MODULE
- NC ZLM** CORRIDOR ZONE LIGHT STATION - CEILING MOUNTED
- MOUNTED IN 1-GANG BOX WITH 1-GANG PLASTER
RING OR A 2-GANG BOX WITH A 1-GANG PLASTER
RING
- ZONE LIGHT TO CONTAIN A ZONE LIGHT MODULE
- TekTONE LI484P5 WITH PM424ZP5 ZONE LIGHT
MODULE
- H** Tek-CARE HUB
- MOUNTED IN 2-GANG ELECTRICAL BOX ABOVE
ACCESSIBLE CEILING
- TekTONE NC331
- NC475** Tek-CARE SERVER, RACK MOUNTED
- TekTONE NC475 (TWO U RACK SPACES)
- NC554** Tek-CARE 15 PORT NETWORK SWITCH
- WALL MOUNTED OR DESK MOUNTED
- TekTONE NC554/15

WIRE LEGEND

- ① COMMUNICATION BUS (C-BUS) HOMERUN TO POE SWITCH.
EACH HUB REQUIRES A DEDICATED HOMERUN
(1) CATEGORY 5e OR BETTER
MAXIMUM LENGTH = 328'
8-PIN CONNECTOR REQUIRED.
- ② HUB STATION WIRING
(1) CATEGORY 5e OR BETTER
MAXIMUM LENGTH = 175'
8-PIN CONNECTOR REQUIRED.
- ③ DOME BUS
(1) CATEGORY 5e OR BETTER
MAXIMUM LENGTH = 50'
6-PIN CONNECTOR REQUIRED.
- ④ PERIPHERAL WIRING
(2) CONDUCTOR #18-#22 STRANDED
MAXIMUM LENGTH = 100'



Ann@PacelliAssociates.com
(518) 414-4302
NYS WBE Certified
PacelliAssociates.com



TekTone 20-Bed Nurse Call System Using Tek-CARE 3000

REVISIONS		DATE
NO	DESCRIPTION	
0	FINAL TO TEK TONE	04/03/20
1	REVISION 01 TO TEK TONE	05/07/20
2	REVISION 02 TO TEK TONE	05/17/20
3	REVISION 03 TO TEK TONE	05/19/20

DRAWING TITLE:
NURSE CALL
SYSTEM
RISER
DIAGRAM

DRAWING NUMBER:

NC2