

### Features

- Electronic DC buzzer
- Tone Off and Tone Test functions
- Emergency and Routine call signaling
- Modular design
- Protected markings

### Description

The Tek-CARE NC110-series Audible-Visual Nurse Call System provides audible and visual call indication without the use of intercom communications. The basic system consists of a master annunciator/control panel, room call stations, emergency call stations, dome lights, zone lights, control unit, and power supply equipment.

Emergency signals are identified at the master station by a rapid flashing of the associated station indicator light, call light, and emergency light; in addition to a rapid pulsating tone. Routine calls are indicated by the steady illumination of the station selector light, call light, and a slow pulsating tone. The tone may be canceled for normal calls by depressing the Tone Off button. Emergency calls take precedence over Routine calls. Call signals can be canceled at point of origin only.

Tone Off and Tone Test functions are push-button selected from the control panel. Indicator lights signal the presence of Emergency or Routine calls.

### Listings



UL® 1069 Listed; UL® File Number E71599.

### Specifications

Dimensions:	Refer to OH200-series housings and OF200-series frames in the IL312 Housings Reference Chart
Power:	24 VAC
Connections:	Screw terminal
Construction:	Extruded Aluminum Panels



NC110/32 Master Panel with OF201 Frame

### Required Components

AM110-series Annunciator Add-On Panels  
NC110-series Master Annunciator Panel  
OH200-series Flush Housing  
OF200-series Frame  
SF-series Call Stations  
IH151NK Power & Control Kit

### Optional Components

PK124K Battery Backup Kit  
PK153 Third Priority Control Unit