




Wiring Diagram for SF380A Visual-Only Patient Station

IL884
Section E
Rev. 2 - 04/2009

NOTE: To reduce drawing complexity, connections shown in drawing do not necessarily reflect physical placement or orientation!

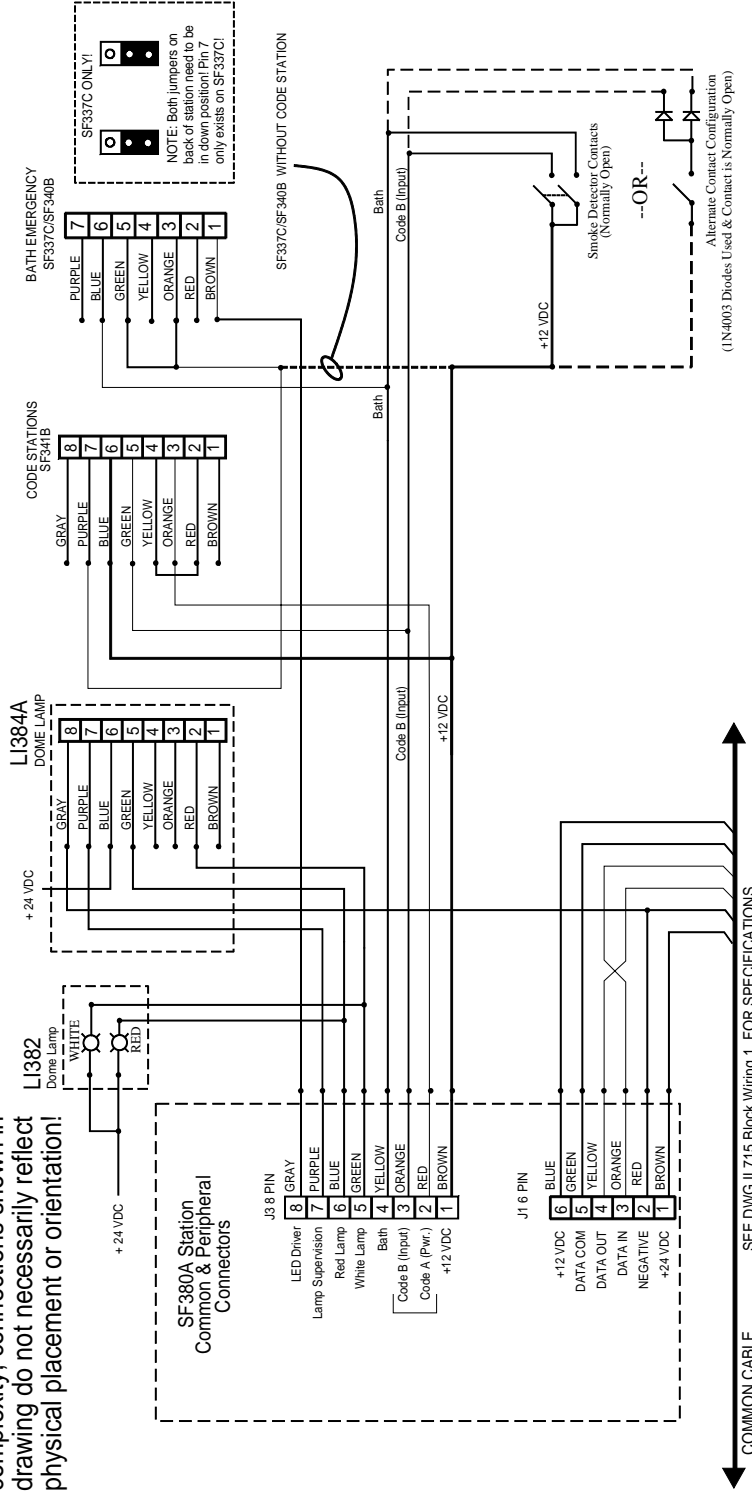

Dipswitch Settings

Switch "ON" = Value	Value
1	1
2	2
3	4
4	8
5	16
6	32
7	See right

DIPSWITCH #7 SETTINGS
ON = LAMP FAULT sensing ENABLED
OFF = LAMP FAULT sensing DISABLED

(1N4003 Diodes Used & Contact is Normally Open)
Alternate Contact Configuration
Smoke Detector Contacts (Normally Open)
--OR--

To NC351(A) (IL715) or NC350C (IL421) Series CE

NOTE: FIRE INDICATION AT THE NURSE MASTER IS SUPPLEMENTARY ONLY.
THE TEK-CARE® NURSE CALL SYSTEM IS NOT INTENDED FOR USE AS A FIRE ALARM/EVACUATION SYSTEM.

Drawing Name & Number: IL884 IL715 IL421 SF380A Wiring Rev/2 01/28/09 1

www.tektone.com

277 Industrial Park Road • Franklin, NC 28734 • Phone (828) 524-9967 • Fax (828) 524-9968

Email: tektone@tektone.net • Technical Assistance Email: teksupt@tektone.net

TekTone®'s quality system is registered by UL® to the ISO 9001:2000 standard. (File #A10766.)