

Tek-CARE150 TekTip: NC150N/NC200N Two-Station Application

We recently received a request from one of our customers for a diagram of two stations sharing a single selective line connection on an NC150/200 System. While TekTone provides a dual station for two bed applications, there are times where the room's architecture does not make this device practical.

Below is a diagram demonstrating how to connect two bed stations in parallel to a single bath station and selective line. Note the two diodes interconnecting the "L" line of the patient stations together. This is important to install as it prevents one station from triggering the other when a call is placed.

(Aug. 1999)

